hCG

Verify Reportable Ranges (Calibration Verification) every 6 months

Verify that the FastPack® IP System is accurate to the limits of the reportable range specified by Qualigen, Inc. by using the FastPack® IP hCG Method Verification Kit.

Low	Write "Low Verifier", the lot number, and your initials on
Target	the peel-off FastPack [®] label and place it here
Mid	Write "Mid Verifier", the lot number, and your initials on
Range	the peel-off FastPack [®] label and place it here
High	Write "High Verifier", the lot number, and your initials on
Target	the peel-off FastPack® label and place it here
mIU/mL to accept the Manufacture above, record the reportable range by	B mIU/mL and the High Verifier result must be > 1000.0 er's Reportable Range. If either result is different than based on actual observed values. The mid value must be ange card. If it is not, repeat the test. If any value is out of Qualigen System Support.
☐ Accept Manufacturer Reportable Range: 1.8 mIU/mL to 1000.0 mIU/mL	
DO NOT Accept. Derived Reportable Range:	

34 65000161 Rev. 008 (04/19)